STERILE SAFETY-VAC 1000+ PRE-VAC WOUND DRAINAGE BOTTLE 1000ml

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REF : SV1000-S

Internally and externally sterile if package is unopened or undamaged.

Re-use of this product will cause the risk of cross infection and unpredictable health threat.

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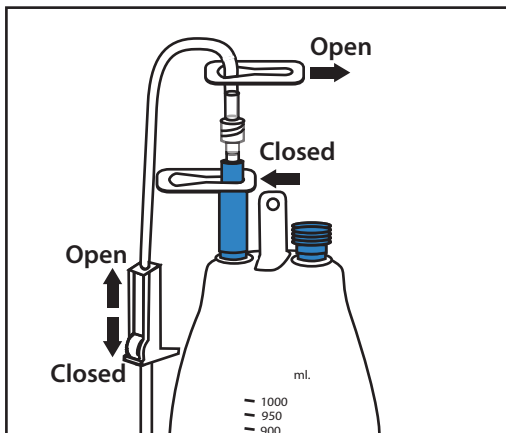
Directions for use

1. Open the sterile pouch and remove the bottle aseptically.
2. Inspect the vacuum indicator to ensure that it is completely compressed. If not, do not use this product.
3. CLOSE CONNECTION TUBE ROLLER CLAMP FIRST. Attach male luer connector of the connecting tube to Female luer of patient drainage device.
4. After checking that all connections are secure, open the slide clamp on the bottle to activate the vacuum.
5. Open the tube roller clamp on the drain-line to begin drainage procedure.
6. If necessary, you may adjust the flow rate during the drainage procedure.
7. Repeat steps 1 to 5 with a new bottle if the bottle is almost full and further drainage is required.
8. Discard the bottle as per your local requirements.

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
STERILE EO

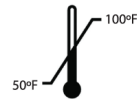LATEX FREE

Distributed Exclusively in the United States by:

Jigsaw Medical

1012 Airpark Dr. Suite 1, Sugar Grove, IL 60554
Tel: 630-587-0019 | Fax: 630-524-9088
Email: Fax@jigsawmed.com
www.jigsawmed.com

Manufactured by:

POLY MEDICURE LIMITED

Plot No. 104-105, HSIDC Industrial Area,
Ballabhgarh-121004, Faridabad, India

Customer Care No. (0129) 4287053

For providing feedback on this product write to customercare@polymedicure.com