STERILE PATHWAY SPIKE SAFETY-VAC 1000+ PRE-VAC WOUND DRAINAGE BOTTLE 1000ml
VÁLVULA ESTERIL SPIKE SAFETY-VAC 1000+ BOTELLA DE DRENAJE DE HERIDAS PRE-VAC 1000ml

REF SV1000-SSP

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GB

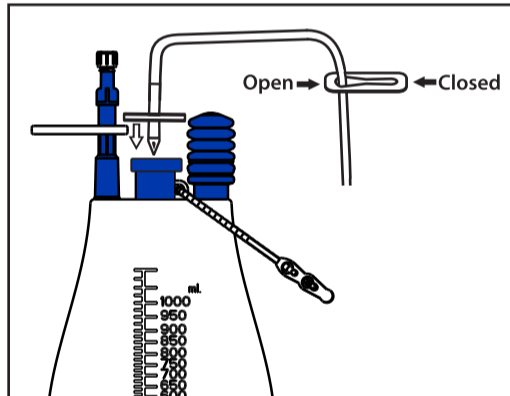
Directions for use

1. Remove the sterile pathway spike bottle from the packaging.
2. Inspect the vacuum indicator to ensure that it is completely compressed. If not, do not use this product.
3. After the appropriate drain line is connected to the catheter, close drain line clamps, then connect to bottle.
4. To use the luer connector for drainage, first remove the cap, tighten the drain line to bottle connector then open slide clamp. Open drain line clamps to activate vacuum.
5. To tap the bottle with a spike/sharp for drainage, first wipe the top of the blue cork in the center of the bottle with alcohol. Push spike thru top of blue cork. Drainage begins once the drain line clamp is opened.
6. If necessary, you may adjust the flow rate during drainage procedure.
7. Repeat steps 1 to 5 with a new bottle if further drainage is required.
8. Discard used bottles as per your local requirements.

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**Packaging
PVC-FREE**

LATEX FREE 50°F 100°F

Distributed Exclusively in the United States by :

1012 Airpark Dr. Suite 1, Sugar Grove, IL 60554
 Tel: 630-587-0019 | Fax: 630-524-9088
 Email: Fax@jigsawmed.com
 www.jigsawmed.com

Manufactured by :
POLY MEDICURE LTD.
 115-116, Sector 59, HSIIDC Industrial Area,
 Faridabad - 121 004 (INDIA)
 Customer Care No. : +91-129-4287053
 For feedback : customercare@polymedicure.com

Rev. 02-20/01

Doc. No. Jigsaw Medical		Date : 30-03-17		Design by	Checked by	Approved by
IFU Design for HPVB 1000 ml Non-Sterile Safety		Size : 210x297 (A4 Sheet) P. Area : 190x277mm				
Color Break ■ □ □ □ Pantone 286 C		Specifications : 1. 2.		Designer	QC Officer	QA Incharge
Rev.	Amendment			Issue No. 03	Scale : 1:1	Sheet 1 of 1
1.	Party address changed					13-7-19
2.	Delete the line Do not use if package is opened or damaged.					27-1-20
3.						
4.						