




Trap Door Re-closable Wound Dressing* Instructions


1

Prepare skin and wound for application of re-closable dressing by preparing the wound by following institutional protocol.

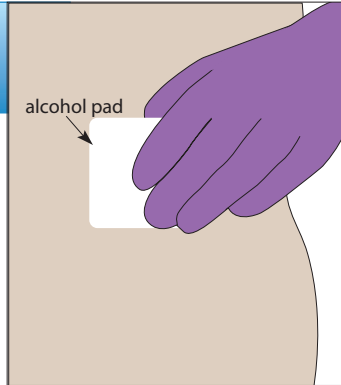


2

Remove the single piece paper backing from the back side of the dressing exposing the underlying adhesive.

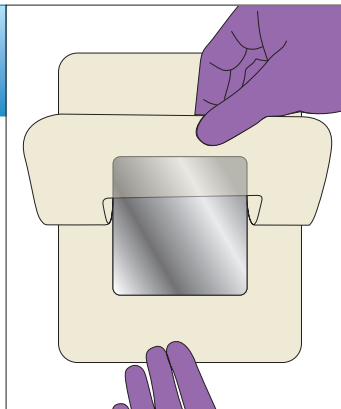


3

Center wound inside the manufactured window and confirm that the wound is within the boundaries of the window before applying the dressing to skin. Apply the dressing to the skin in a top to bottom or edge to edge fashion.

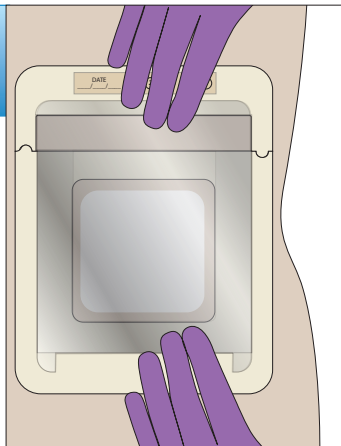

4

Slowly remove both portions of the two-piece paper backing with one hand while applying counter-pressure to the base film with a free hand. Begin with top portion with the Dressing Change Sticker setting it aside for use in Step 8.

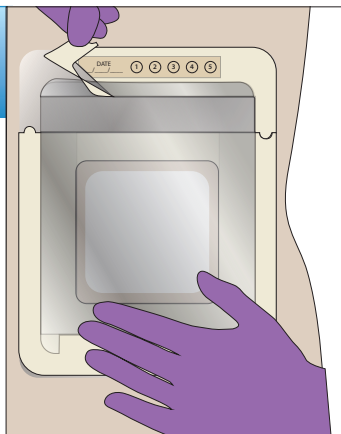


IMPORTANT:

The Trap Door dressing when fully adhered is waterproof and breathable. Do not immerse in water and avoid direct flow of shower. Do not use the dressing as a primary wound closure device. Stop all bleeding prior to applying the dressing.


5

Next slowly remove the bottom portion of the backing taking care to avoid unwanted bubbles and/or wrinkles in the film. As shown in the illustration.

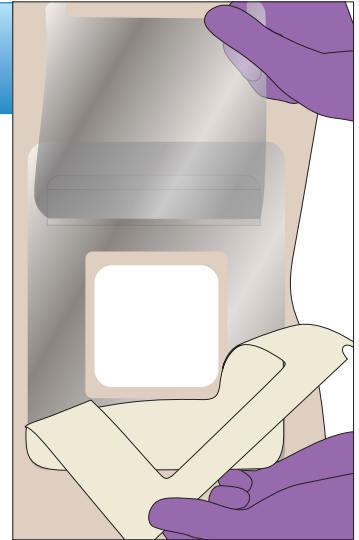


6

Remove the remaining clear plastic protective layer from the underside of the wound cover as shown in the illustration.

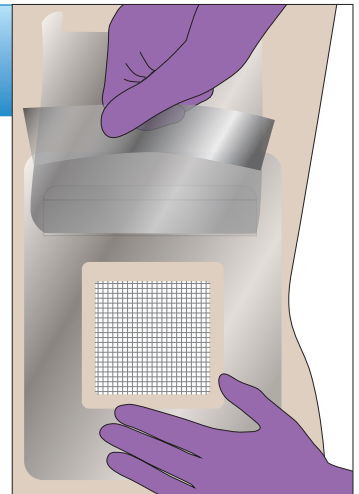

7


Position any gauze or foam as directed over the wound taking care not to exceed the edges forming the window in the base film. As shown in the illustration.

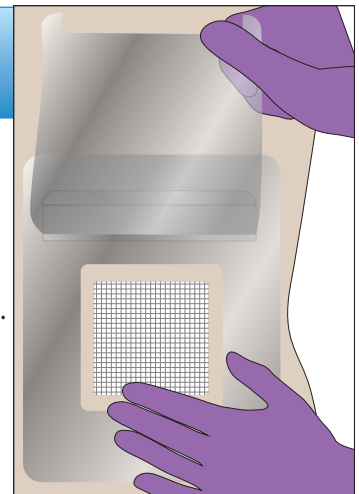

Instructions continue on backside...


8

Close the cover insuring that all edges are completely adhered to the base layer. Avoid contact of the adhesive with the wound or any material such as foam or gauze that is used to cover the wound. Smooth out any wrinkles and or bubbles to insure a safe closure.

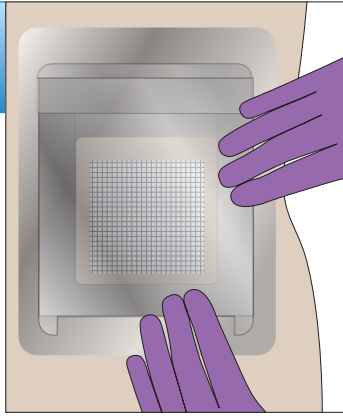


9

Write the re-closable dressing application date on the Change Sticker located on the top portion of backing which was set aside in Step 4. Peel Change Sticker off of the top backing paper.

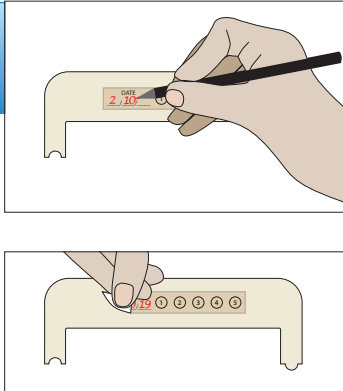


10

Apply Change Sticker to the re-closable dressing as shown.

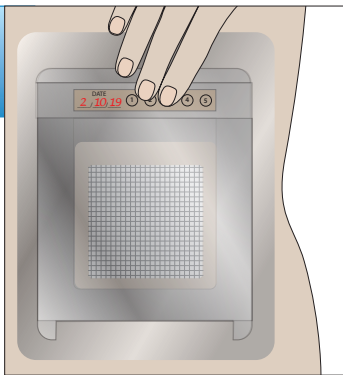

11

To change any product used to cover the wound such as foam or gauze open the cover layer. . . . Remove the old gauze or foam clean wound, apply new gauze or foam as directed and close cover. Use sticker provided on the change chart to note each change of any gauze or foam used to cover the wound.

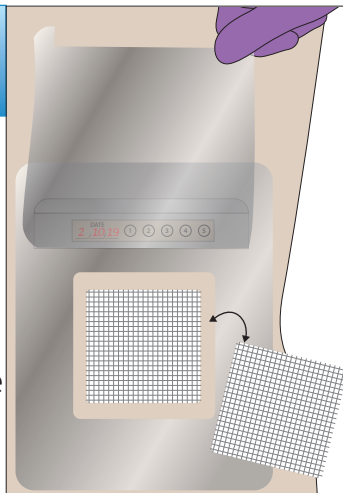

12

Each time the wound is accessed write the date on the enclosed Change Chart then peel off the corresponding circular X sticker to the right of the date.

CHANGE CHART

Dressing Application Date
2 / 10 / 19

After 7 days of continuous wear or 5 gauze/foam changes or if the dressing becomes detached, shows signs of peeling and or if skin becomes irritated remove dressing and replace if necessary.

Change Dates **Removable Stickers**

1. ___/___/___ ○

2. ___/___/___ ○

3. ___/___/___ ○

4. ___/___/___ X


5. ___/___/___ X

After 7 days of wear or if dressing becomes detached, shows signs of peeling or and loss of adhesion remove dressing and replace as directed. Please visit our website at www.jigsaw.com for additional information.

Jigsaw Medical


13

Place the "x" sticker on the re-closable wound dressing date sticker.

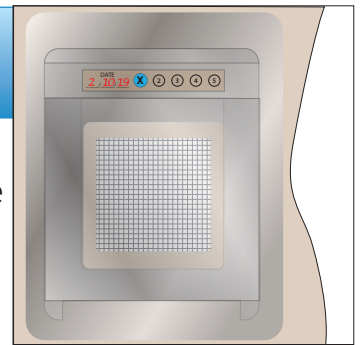


14

After 7 days of continuous wear or 5 gauze/foam changes or if the dressing becomes detached, shows signs of peeling and or if skin becomes irritated: Completely remove and discard re-closable dressing. After removal replace as directed. See below for recommendations for best removal of re-closeable dressing.

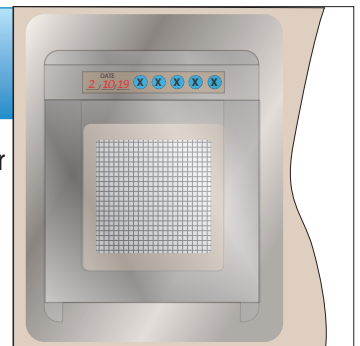

15

Remove the dressing by gently supporting the skin then gripping the edge and slowly peeling from the skin in the direction of hair growth. Avoid damaging skin by peeling back rather than up from the skin. Use of a medical adhesive solvent may help assist with dressing removal.

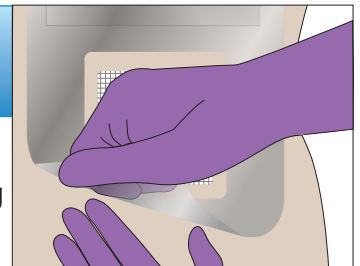

Jigsaw Medical™

Solving the healthcare puzzle one product at a time

*Patent Pending Scan for Instructions
For additional information visit our website.
www.jigsawmed.com

1012 Airpark Dr., Suite 1, Sugar Grove, IL 60554
Tel: 630-587-0019 | Fax: 630-524-9088
Email: info@jigsawmed.com

